IMPORTANT: The Eye Care Professional should provide the patient with the appropriate Instructions that pertain to the patient's prescribed lenses and should discuss with the patient the risks associated with wearing and caring of contact lenses.

ACUVUE®

Brand Contact Lenses

Package Insert

CE
2797

English

IMPORTANT: Please read carefully and keep this information for future use

English

Important: Please read carefully and keep this information for future use. This Package Insert is intended for the Eye Care Professional but should be made available to patients upon request. This insert contains important information about the correct usage of ACUVUE® Brand Contact Lenses, which are listed in Table 1 and contains information about adverse reactions and contraindications.

Johnson & Johnson Medical Ltd provides ACUVUE® Brand Contact Lenses Patient Instruction Guides that contain further information for contact lens wearers. The Eye Care Professional should provide the patient with the Patient Instruction Guide that pertains to the patient's prescribed lens. Copies are available for download at local Acuvue website.

Table 1

Lens type and Brand name	Intended Use & Wear Schedule			Material	Polymer/water content (%)	Packaging Solution
	Daily Wear - Daily disposable	Daily wear Frequent replacement	Extended wear			
ACUVUE® Brand Spherical Contact Lenses - Visibility Tinted with UV Blocker						
ACUVUE®2 BRAND CONTACT LENSES		⊙	⊙	etafilcon A	42/58	①
1-DAY ACUVUE® Brand Contact Lenses	⊙			etafilcon A	42/58	①
1-DAY ACUVUE® MOIST BRAND CONTACT LENSES WITH LACREON®	⊙			etafilcon A	42/58	③
ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS		⊙	⊙	senofilcon A ④	62/38	②
ACUVUE OASYS® Brand Contact Lenses with HydraLuxe®	⊙			senofilcon A ④	62/38	②
1-DAY ACUVUE® Trueye® Brand Contact Lenses with HYDRACLEAR® 1	⊙			narafilcon A ④	54/46	②
1-DAY ACUVUE® DEFINE® Brand Contact Lenses with LACREON®	⊙			etafilcon A	42/58	③
ACUVUE® VITA® Brand Contact Lenses		⊙		senofilcon C ④	59/41	②
ACUVUE® OASYS Contact Lenses with Transitions™		⊙		senofilcon A ④ ⑤	62/38	②
ACUVUE® Brand Contact Lenses for ASTIGMATISM - Visibility Tinted with UV Blocker						
1-DAY ACUVUE® MOIST BRAND CONTACT LENSES FOR ASTIGMATISM WITH LACREON®	⊙			etafilcon A	42/58	③
ACUVUE® VITA® Brand Contact Lenses for ASTIGMATISM		⊙		senofilcon C ④	59/41	②
ACUVUE® OASYS® Brand Contact Lenses for ASTIGMATISM with HYDRALUXE®	⊙			senofilcon A ④	62/38	②
ACUVUE OASYS® Brand Contact Lenses for ASTIGMATISM with HYDRACLEAR® PLUS		⊙	⊙	senofilcon A ④	62/38	②
ACUVUE® Brand Contact Lenses for PRESBYOPIA - Visibility Tinted with UV Blocker						
1-DAY ACUVUE® MOIST Brand Contact Lenses MULTIFOCAL	⊙			etafilcon A	42/58	③
ACUVUE OASYS® Brand Contact Lenses for PRESBYOPIA with HYDRACLEAR® PLUS		⊙	⊙	senofilcon A ④	62/38	②

Key

Packaging Solution: ① Borate buffered saline ② Borate buffered saline with methyl ether cellulose ③ Borate buffered saline with povidone.

Material content: ④ Lens material contains silicone and meets Class 1 UV absorbing standards with transmissibility of less than 1% UVB (280 -315 nm) and 10% UVA (316-380nm) radiation. All etafilcon A products meet Class 2 UV absorbing standards with transmissibility of less than 5% UVB and 50% UVA radiation. ⑤ Lens material contains a photochromic additive which dynamically absorbs visible light in the range from 380 nm to 780 nm to a minimum 84% transmittance in the inactivated (closed) state and to a minimum of 23% transmittance in the activated (open) state dependent on the lens thickness and the level of absorbed UV and high energy visible (HEV) radiation.

Symbols Key

The following symbols may appear on the label or packaging.

SYMBOL	DEFINITION
	Caution, Consult Instructions for Use
	Manufacturer
	Date of Manufacture
	Use By Date (expiration date)
	Batch Code
	Sterilized Using Steam Heat
	Do Not Use If Package Is Damaged
	Do Not Re-Use (Single Use)
DIA	Diameter
BC	Base Curve
D	Dioptre (lens power)
CYL	Cylinder
AXIS	Axis
MAX ADD	Highest near addition that can be corrected
LOW / L	"Low" near ADD
	Store Away from Direct Sunlight

SYMBOL	DEFINITION
MID / M	"Medium" near ADD
HGH / H	"High" near ADD
S_H	NATURAL SHIMMER™
S_P	NATURAL SPARKLE™
V	VIVID STYLE
A	ACCENT STYLE
N	NATURAL SHINE™
RB	Radiant Bright™
RC	Radiant Charm™
1-Month	1-Month Replacement
	Quality System Certification Symbol
 UV-Blocking	UV-Blocking
	Fee Paid for Waste Management
	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a licensed practitioner
	Lens Orientation Correct
	Lens Orientation Incorrect (Lens Inside Out)
	"Identification mark" for paper containers and wrapping
	"Identification mark" for composite materials
	Authorised Representative in the European Community

CONTENT

ACUVUE® Brand Contact Lenses are provided in sterile individual blister packages, with the lens immersed in borate buffered saline solution (see **Table 1** for Packaging Solution). **DO NOT USE** if the sterile blister package is opened or damaged.

INTENDED USE

ACUVUE® Brand Spherical Contact Lenses

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes who may have 1.00D or less of Astigmatism.

ACUVUE® OASYS with Transitions™ are photochromic contact lenses also indicated for the attenuation of bright light as they contain a photochromic additive which dynamically absorbs visible light.

1-DAY ACUVUE® DEFINE® Brand Contact Lenses with LACREON® are also intended to enhance or alter the appearance of the eye.

ACUVUE® Brand Contact Lenses for Astigmatism

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes who may have astigmatism.

ACUVUE® Brand Contact Lenses for Presbyopia

Are intended for Daily Wear or Extended Wear, as shown in **Table 1**, for the optical correction of distance and near vision in presbyopic, phakic or aphakic persons with non-diseased eyes who have 0.75D or less of astigmatism.

Additionally, ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS are also indicated for therapeutic use as a bandage lens for the following acute and chronic ocular conditions:

- For corneal protection in lid and corneal abnormalities such as entropion, trichiasis, tarsal scars and recurrent corneal erosion. In addition, they are indicated for protection where sutures or ocular structure malformation, degeneration or paralysis may result in the need to protect the cornea from exposure or repeated irritation.
- For corneal pain relief in conditions such as bullous keratopathy, epithelial erosion and abrasion, filamentary keratitis, and post-keratoplasty.
- For use as a barrier during the healing process of epithelial defects such as chronic epithelial defects, corneal ulcer, neurotrophic and neuroparalytic keratitis, and chemical burns.
- For post surgical conditions where bandage lens use is indicated such as post refractive surgery, lamellar grafts, corneal flaps, and additional ocular surgical conditions.

- For structural stability and protection in piggy back lens fitting where the cornea and associated surfaces are too irregular to allow for corneal rigid gas permeable (RGP) lenses to be fitted. In addition, the use of the lens can prevent irritation and abrasions in conditions where there are elevation differences in the host/graph junction or scar tissue. ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS when prescribed for therapeutic use may be worn for Daily Wear or Extended Wear.

ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR PLUS when prescribed for therapeutic use may be worn for Daily Wear or Extended Wear.

All ACUVUE® Brand Contact Lenses have UV Blocking to help provide protection against transmission of harmful UV radiation to the cornea and into the eye.

WARNING: UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. You should continue to use UV absorbing eyewear as directed.

Note: Long-term exposure to UV radiation is one of the risk factors associated with cataracts. Exposure is based on a number of factors such as environmental conditions (altitude, geography, cloud cover) and personal factors (extent and nature of outdoor activities). All ACUVUE® Brand Contact Lenses have UV blocking to help provide protection against transmission of harmful UV radiation to the cornea and into the eye. However, clinical studies have not been done to demonstrate that wearing UV blocking contact lenses reduces the risk of developing cataracts or other eye disorders. Consult your Eye Care Professional for more information.

WEAR SCHEDULE

The wearing and replacement schedule should be determined by the Eye Care Professional based upon the patient's history and their ocular examination, as well as the practitioner's experience and clinical judgment. Patients may tend to over wear the lenses initially. The Eye Care Professional should emphasise the importance of adhering to the initial maximum wearing schedule. Regular checkups, as determined by the Eye Care Professional, are also extremely important.

Daily Wear - Daily Disposable

ACUVUE® Brand Contact Lenses prescribed for Daily Wear - Daily Disposable (less than 24 hours, while awake) as shown in **Table 1**, are intended to be worn once on a daily disposable basis and are to be discarded upon removal. **When used in this way, no cleaning or disinfection is required.** Start each wearing period with a fresh new lens.

Daily Wear - Frequent Replacement

All ACUVUE® VITA® Brand Contact Lenses prescribed for Daily Wear – Frequent Replacement (less than 24 hours, while awake) as shown in **Table 1**, are to be discarded and replaced every month.

All other ACUVUE® Brand Contact Lenses prescribed for Daily Wear - Frequent Replacement, are to be discarded and replaced every two weeks.

All ACUVUE® Brand Contact Lenses as shown in **Table 1**, under the heading of Daily Wear - Frequent Replacement are to be cleaned, rinsed and disinfected each time the lens is removed using a chemical disinfection system only.

Extended Wear

ACUVUE® Brand Contact Lenses prescribed for Extended Wear, (greater than 24 hours, including while asleep) as shown in **Table 1**, may be used continuously for up to 7 days/6 nights and should be discarded upon removal. When used in this way, **no cleaning or disinfection is required. Start each wearing period with a fresh new lens.**

It is recommended that the new contact lens wearer first be evaluated on a Daily Wear - Frequent Replacement schedule. If in the opinion of the Eye Care Professional, the patient is determined to be an acceptable Extended Wear candidate, the Eye Care Professional is encouraged to determine a wear schedule based upon the response of the patient. Not all patients can achieve the maximum wear time.

Once removed, it is recommended that the lens remains out of the eye for a period of rest overnight or longer. The Eye Care Professional should examine the patient during the early stages of Extended Wear.

CONTRAINDICATIONS

DO NOT USE ACUVUE® Brand Contact Lenses when any of the following conditions exist, when prescribing for refractive ametropia use.

- Acute or subacute inflammation or infection of the anterior chamber of the eye.
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids.
- Severe insufficiency of lacrimal secretion (dry eye).
- Corneal hypoesthesia (reduced corneal sensitivity).
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient, such as mercury or Thimerosal in a solution which is to be used to care for the lenses prescribed on a frequent replacement wear schedule.
- Any active corneal infection (bacterial, fungal, protozoal or viral).
- If eyes become red or irritated.

For THERAPEUTIC USE, the Eye Care Professional may prescribe ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS, to aid in the healing process of certain ocular conditions, which may include those cited above.

WARNINGS

(Daily Wear = less than 24 hours, while awake; Extended Wear = greater than 24 hours, including while asleep).

Proper use and care of contact lenses and lens care products, including lens cases, are essential for safe use. Problems from wearing contact lenses or using lens care products could result in serious injury to the eye.

Patients should be advised of the following warnings pertaining to contact lens wear :

- ACUVUE® Brand Daily Disposable Contact Lenses are prescribed for daily wear and are for single use. Studies have shown that daily disposable soft contact lens wear reduces the risk of some complications including discomfort and inflammation that are associated with lens care and handling, and reuse can put you at greater risk of these problems.
- Problems with contact lenses or lens care products could result in serious injury to the eye. Patients should be cautioned that proper use and care of contact lenses and lens care products, including lens cases, are essential for the safe use of these products.
- Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision.
- Studies have shown that the risk of eye problems, including ulcerative keratitis is greater for extended wear contact lens users than for daily wear users.

- When daily wear users wear their lenses overnight (outside the intended indication), the risk of eye problems, including ulcerative keratitis is greater than among those who do not wear them overnight[†].
- The overall risk of eye problems, including ulcerative keratitis may be reduced by carefully following directions for lens care, including cleaning the lens case.
- Studies have shown that the risk of eye problems, including ulcerative keratitis among contact lens users who smoke is greater than among non-smokers.
- **Do not** expose contact lenses to **water** during swimming, other water sports or bathing as this **could** increase the risk of serious eye infection from microorganisms which could **lead to vision loss**. If lenses have been submersed in water, the patient should discard and replace them with a new pair. The Eye Care Professional should be consulted for recommendations regarding wearing lenses during any activity involving water.

Specific Instructions for Use and Warnings for Lenses Prescribed for Daily Wear Frequent Replacement and Extended Wear:

- Do not reuse or “top-off” old solution left in your lens case since solution reuse reduces effective lens disinfection and could lead to severe infection, vision loss, or blindness. “Topping-Off” is the addition of fresh solution to solution that has been sitting in your case.
- Discard any remaining solution after the recommended time period indicated on the bottle of multi-purpose solution used for disinfecting and soaking your contact lenses. The discard date refers to the time you can safely use the contact lens care product after the bottle has been opened. It is not the same as the expiration date, which is the last date that the product is still effective before it is opened.
- Using multi-purpose solution beyond the discard date could result in contamination of the solution and can lead to severe infection, vision loss, or blindness. To avoid contamination, DO NOT touch the tip of the container to any surface. Replace cap after using. DO NOT transfer to other bottles or containers.
- To adequately disinfect the lenses, rub and rinse the lenses according to the recommended lens rubbing and rinsing times in the labelling of the multi-purpose solution. Rub and rinse lenses for the recommended amount of time to help prevent serious eye infections.
- Never use water, saline solution, or rewetting drops to disinfect the lenses. These solutions will not disinfect the lenses. Not using the recommended disinfectant can lead to severe infection, vision loss, or blindness.
- Replace the lens case according to the directions given by the Eye Care Professional or the manufacturer’s labelling that accompanies the case.

[†] New England Journal of Medicine, September 21, 1989, 321 (12), pp. 773-783

- Empty and clean contact lens cases with digital rubbing using fresh, sterile disinfecting solutions/contact lens cleaner. Never use water. Cleaning should be followed by rinsing with fresh, sterile disinfecting solutions (never use water) and wiping the lens cases with fresh, clean tissue is recommended. Never air-dry or recap the lens case lids after use without any additional cleaning methods. If air-drying, be sure that no residual solution remains in the case before allowing it to air-dry.
- Do not store your lenses or rinse your lens case with water or any non-sterile solution. Only use fresh multi-purpose solution so you do not contaminate your lenses or lens case. Use of non-sterile solution can lead to severe infection, vision loss, or blindness.

If patients experience eye discomfort, excessive tearing, vision changes, redness of the eye or other problems are experienced, they should be instructed to immediately remove their lenses and the wearer should promptly contact their Eye Care Professional. It is recommended that contact lens wearers should see their Eye Care Professional routinely as directed.

PRECAUTIONS

What to do if problems occur

PLEASE ADVISE YOUR PATIENT TO REMOVE CONTACT LENSES IMMEDIATELY AND SEEK ADVICE FROM THEIR EYE CARE PROFESSIONAL.

Special Precautions for Eye Care Professionals

Due to the small number of patients enrolled in clinical investigation of lenses, all refractive powers, design configurations or lens parameters available in the lens material are not evaluated in significant numbers. Consequently, when selecting an appropriate lens design and parameters, the Eye Care Professional should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen transmissibility, wettability, central and peripheral thickness and optic zone diameter.

The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive correction; therefore, the continuing ocular health of the patient and lens performance on the eye should be carefully monitored by the prescribing Eye Care Professional.

- Due to the reduction in light transmittance with cosmetically tinted lenses some patients may experience visual symptoms while wearing 1-DAY ACUVUE® DEFINE® Brand Contact Lenses with LACREON®. In addition, some patients may experience peripheral awareness due to the opaque iris pattern.

- Patients who wear the ACUVUE® Brand Contact Lenses to correct presbyopia using Monovision may not achieve the best corrected visual acuity for either far or near vision. Visual requirements vary with the individual and should be considered when selecting the most appropriate type of lens for each patient.
- Studies have shown that the risk of eye problems, including ulcerative keratitis is greater for extended wear contact lens users than for daily wear users.
- Fluorescein, a yellow dye, should not be used while the lenses are on the eyes. The lenses absorb this dye and become discolored. Whenever fluorescein is used in eyes, the eyes should be flushed with a sterile saline solution that is recommended for in-eye use.
- Eye Care Professionals should instruct patients to always have a functional pair of spectacles with a current prescription available to use if they become unable to wear contact lenses, or in circumstances where contact lens wear is not advised.
- Eye Care Professionals should instruct the patient to remove the lenses immediately if the eyes become red or irritated.
- Eye Care Professionals should inform the patient that there is no data available on the safety driving performance with ACUVUE® OASYS with Transitions™ contact lenses for individuals age 50 and older who may have crystalline lens opacities.
- Eye Care Professionals should carefully instruct patients about the following care regimen and safety precautions.

Handling Precautions

- Before leaving the Eye Care Professional's office, the patient should be able to put on and promptly remove lenses or should have someone else available who can put on and remove the lenses for him or her.
- **DO NOT** use if the sterile blister package is opened or damaged.
- **Always** wash hands with soap & warm water, rinse and dry hands with a clean lint-free towel before handling lenses to reduce the chance of getting an infection. Do not get cosmetics, lotions, soaps, creams, deodorants or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup.
- Do not touch contact lenses with the fingers or hands if the hands are not free of foreign materials, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to the eye.
- Carefully follow the handling, insertion, removal, cleaning, disinfecting, storing and wearing instructions in the "Patient Instruction Guide" for ACUVUE® Brand Contact Lenses and those prescribed by the Eye Care Professional.
- **Always** handle lenses carefully and avoid dropping them.
- Never use tweezers or other tools to remove lenses from the lens container. Carefully remove the lens by sliding it up the side of the container.
- Do not touch the lens with fingernails.
- Close supervision is necessary for the Therapeutic use of all ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS. Ocular medications used during treatment with a bandage lens should be closely monitored by the Eye Care Professional. In certain ocular conditions, only the Eye Care Professional will insert and remove the lenses. In these cases, patients should be instructed not to handle the lenses themselves.

Lens Wearing Precautions

- If the lens sticks (stops moving) on the eye, follow the recommended directions in "Care for a Sticking Lens". The lens should move freely on the eye for the continued health of the eye. If non-movement of the lens continues, the patient should be instructed to immediately consult his or her Eye Care Professional.
- Never wear lenses beyond the period recommended by the Eye Care Professional.
- If aerosol products, such as hair spray, are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Avoid all harmful or irritating vapours and fumes while wearing lenses.
- Never allow anyone else to wear your lenses. Sharing lenses greatly increases the chances of eye infections.
- The patient should be advised to never rinse the lenses in water from the tap. Tap water contains many impurities that can contaminate or damage the lenses and may lead to eye infection or injury.
- Be aware that wearing a darkened ACUVUE® OASYS with Transitions™ lens on only one eye is not recommended because it may cause disturbances in the patient's ability to accurately judge depth and the motion of objects. It may also create a cosmetic concern. Therefore, unilateral vision correction is not recommended for these contact lenses.
- Be aware that ACUVUE® OASYS with Transitions™ contact lenses are not intended for use as protection against artificial light sources, such as sun lamps, lasers, etc.
- The patient should be advised to never stare directly at the sun or at an eclipse with or without ACUVUE® OASYS with Transitions™ contact lenses.
- After the recommended wearing schedule, always discard lenses worn as prescribed by the Eye Care Professional.

Solution Precautions

- Different solutions cannot always be used together and not all solutions are safe for use with all lenses. Use only recommended solutions.
- The patient should not change solutions without consulting their Eye Care Professional.
- Never use solutions recommended for rigid gas permeable (RGP) contact lenses.
- Always use fresh, unexpired lens care solutions and lenses.
- Always follow directions in the package inserts for the use of contact lens solutions.
- Use only a chemical (not heat) lens care system. Use of a heat (thermal) care system can damage ACUVUE® Brand Contact Lenses.
- Sterile unpreserved solutions, when used, should be discarded after the time specified in the directions.
- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.

- Always keep the lenses completely immersed in the recommended storage solution when the lenses are not being worn (stored). Prolonged periods of drying will reduce the ability of the lens surface to return to a wettable state. Follow the lens care directions in "Care for a Dried Out (Dehydrated) Lens" if lens surface does become dried out.

Lens Case Precautions

- Lens cases can be a source of bacterial growth and require proper use, cleaning and replacement at regular intervals as recommended by the lens case manufacturer or Eye Care Professional.
- Always store individual unopened ACUVUE® OASYS with Transitions™ blisters out of direct sunlight.
- Always store worn ACUVUE® OASYS with Transitions™ contact lenses in the lens case and out of direct sunlight.

Other Topics to Discuss with Patients

- Always contact the Eye Care Professional before using any medicines or eye drops in the eyes.
- Certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilisers and those for motion sickness may cause dryness of the eye, increased lens awareness or blurred vision. Should such conditions exist, proper remedial measures should be prescribed.
- Oral contraceptive users could develop visual changes or changes in lens tolerance when using contact lenses. Patients should be cautioned accordingly.
- Do not change lens type (e.g. brand name, etc.) or parameters (e.g. diameter, base curve, lens power, etc.) without consulting the Eye Care Professional.
- Instruct patients to always confirm the lens parameters printed on the multi-pack and on the individual lens package match their prescription. If there is a mismatch the patient should not use the product.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of the patient's eyes. The patient should be instructed as to a recommended follow-up schedule.

Who Should Know That the Patient is Wearing Contact Lenses?

- Patients should inform all doctors (Health Care Professional) about being a contact lens wearer.
- Always inform the employer of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that the patient not wear contact lenses.

ADVERSE REACTIONS

The patient should be informed that the following problems may occur when wearing ACUVUE® Brand Contact Lenses:

- The eye may burn, sting and/or itch.
- There may be less comfort than when the lens was first placed on the eye.

- There may be a feeling of something in the eye (foreign body, scratched area).
- There may be the potential for some temporary impairment due to peripheral infiltrates, peripheral corneal ulcers and corneal erosion. There may be the potential for other physiological observations, such as local or generalized oedema, corneal neovascularization, corneal staining, injection, tarsal abnormalities, iritis and conjunctivitis, some of which are clinically acceptable in low amounts.
- There may be excessive watering, unusual eye secretions, or redness of the eye.
- Poor visual acuity, blurred vision, rainbows or haloes around objects, photophobia, or symptoms of eye dryness may also occur if the lenses are worn continuously or for too long a time.

The patient should be instructed to conduct a simple 3-part self-examination at least once a day. They should ask themselves:

- How do the lenses feel on my eyes?
- How do my eyes look?
- Have I noticed a change in my vision?

If the patient reports any problems, he or she should be instructed to **IMMEDIATELY REMOVE THE LENS**. If the discomfort or problem stops, the patient should then look closely at the lens. If the lens is in any way damaged, the patient **SHOULD NOT** put the lens back on the eye. The patient should discard the lens and apply a new fresh lens on the eye.

If the lens has dirt, an eyelash, or foreign body on it, or the problem stops and the lens appears undamaged, he or she should be instructed to dispose of the lens and apply a new fresh lens. If the problem continues, the patient **SHOULD NOT** put the lens back on the eye but **IMMEDIATELY CONSULT HIS OR HER EYE CARE PROFESSIONAL**.

The patient should also be instructed **NOT** to use a new lens as self-treatment for the problem. The patient should be advised that when any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. The patient should be instructed to seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

During therapeutic use of ACUVUE OASYS® Brand Contact Lenses with HYDRACLEAR® PLUS an adverse effect may be due to the original disease or injury or may be due to the effects of wearing a contact lens. There is a possibility that the existing disease or condition might become worse when a soft contact lens for therapeutic use is used to treat an already diseased injured eye. The patient should be instructed to avoid serious eye damage by contacting the Eye Care Professional **IMMEDIATELY** if there is an increase in symptoms while wearing the lens.

Lens care directions

When lenses are dispensed, the Eye Care Professional should provide the patient with appropriate and adequate warnings and instructions in accordance with the individual patient's lens type and wearing schedule. The Eye Care Professional should recommend an appropriate care system tailored to the patient's individual requirements.

Failure to follow the correct lens care regime may result in serious injury to the eye, as described in the section entitled – **“Warnings”**

For complete information concerning contact lens handling, care, cleaning, disinfecting and storage, refer to the Patient Instruction Guide for ACUVUE® Brand Contact Lenses.

When ACUVUE® Brand Contact Lenses are recommended for frequent replacement, as shown in **Table 1**, the lenses must be cleaned and disinfected following removal before re-use. The lenses may be disinfected using a chemical disinfection system only (e.g. multi-purpose or hydrogen peroxide system).

The Eye Care Professional should review with the patient lens care directions, including basic information on lens case cleaning as well as specific instructions on the lens care regimen recommended for the patient. Since some lens materials contain silicone, as shown in **Table 1**, the wettability may differ when different lens care products are used.

Care for a sticking (non-moving) lens

If the lens sticks (stops moving), the patient should be instructed to apply a few drops of a recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after a few minutes, the patient should immediately consult the Eye Care Professional.

Care for a dried out (dehydrated) lens

If any ACUVUE® lens is off the eye for a prolonged period, its surface may become dry and gradually become non-wetting. If this should occur, discard the lens and use a new one.

EMERGENCIES

The patient should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should: **FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYE CARE PROFESSIONAL OR VISIT A HOSPITAL CASUALTY DEPARTMENT WITHOUT DELAY.**

REPORTING OF ADVERSE REACTIONS

All serious adverse experiences and adverse reactions observed in patients wearing ACUVUE® Brand Contact Lenses or experienced with the lenses should be reported to:

(Johnson & Johnson Middle East) Inc.
Dubai Health Care City, Building 14, Level 7
Mohamed Bin Rashid Al Maktoum Academic Medical Centre,
PO Box 505080, Dubai, United Arab Emirates
Tel: +971(4)4297747 Fax: +971(4)4297450
acuvuearabia@its.jnj.com
www.acuvuearabia.com

Additional information

For additional information concerning ACUVUE® Brand Contact Lenses and to order your free copy of ACUVUE® Brand Contact Lenses Patient Instruction Guides, please contact the address shown above.

Manufactured by:

Please refer to carton for site of manufacture

USA:

Johnson & Johnson Vision Care, Inc.
7500 Centurion Parkway
Jacksonville
Florida, 32256
USA

IRELAND:


Johnson & Johnson Vision Care (Ireland)
The National Technology Park
Limerick
Ireland

EU Authorised Representative:

AMO Ireland
Block B, Liffey Valley Office Campus,
Quarryvale, Co. Dublin, Ireland

www.acuvuearabia.com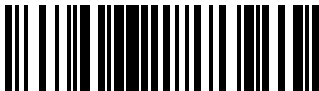

IN000372

**ACUVUE®
Contact Lenses Package Insert
(English)
IN000372**

©Johnson & Johnson Medical Limited. 2019

ACUVUE®, 1-DAY ACUVUE®, HYDRACLEAR®, ACUVUE OASYS®, HydraLuxe®, VITA®, TruEye®, DEFINE® and LACREON® are trademarks of Johnson & Johnson Medical Limited.

Transitions™ is a trademark of Transitions Optical, Inc., used under license by TOL and JJVCI.

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